



[Top Longmax Products Co., Ltd.](#)

TOP-505 Correction & Position Wheelchair



Design and specifications subject to change without notice.

## Preface

Thank you for buying our product.

Before using your wheelchair,  
firstly assemble, operate  
and maintain this wheelchair  
according to this manual,  
pay special attention to  
adjustment and safety sections.

## TABLE OF CONTAIN

### **TOP-505 Correction & Position Wheelchair**

1. [Specification](#)
2. [User's Manual](#)
3. [Table Assembly](#)

# 1. Specification

## **Correction & Positioning Wheelchair**

- ◆ Aluminum Alloy Frame
- ◆ Fully 90 Degrees Reclining Backrest
- ◆ Adjustable Headrest
- ◆ Vertical High Adjustable Footrest
- ◆ Solid Caster 5" x 1"
- ◆ Quick Release 12-1/2" x 2-1/4" Pneumatic Rear Wheels
- ◆ Safety Special Seat Belt & Leg Strap
- ◆ Adjustable Anti-Tipper
- ◆ Available Seat Width : 12" . 13" . 14" . 15" . 16"
- ◆ Available For Fold Backrest

## 2. User's Manual

- 1) Release the winding knob, and adjust the headrest backward and forward to the right position, then fasten up the knob. ([Picture A](#))
- 2) Release the winding knob, and adjust the headrest up and down to the right position, then fasten up the knob. ([Picture B](#))
- 3) Grasp the right and left finger lever of the recliner, adjust the seat and back cushion to the right position, then release the finger lever to fix the reclining position. ([Picture C](#))
- 4) Use the right hex key to release the fastening block of footrest, adjust it to the right position, then fasten up the fastening block to fix it. ([Picture D](#))
- 5) Press down the handle to apply the brake, and push up the handle to release the brake. ([Picture E](#))
- 6) Press down the snap cover of the hub to release (take off) the rear wheel. Reverse the procedure for fasten the rear wheel ([Picture F](#))
- 7) Release the winding knob of the side paddles of the hips & thighs under the seat cushion, adjust to the right seat width, then fasten up the knob, also release the winding knob on the side paddles for adjust to the right height, then fasten it. ([Picture G](#)) same procedure to the center paddle of crotch.
- 8) Release the winding knob of the chest side paddles behind the back cushion, adjust to the right body's width, then fasten up the knob, also release the winding knob on the front side paddles for adjust to the right depth, then fasten it. ([Picture H](#))
- 9) Fasten the seat belt before using the wheelchair. ([Picture I](#))
- 10) The procedure of retracting the wheelchair frame:
  - a. Untied the seat belt. ([Picture J](#))
  - b. Pull up the back cushion with force. ([Picture K](#))
  - c. Pull up the seat cushion with force. ([Picture L](#))
  - d. Push the plastic footplates outward. ([Picture M](#))

- e. Push down the back at the joint and let the handles to downward. (If the chair with fold back) ([Picture N](#))
- f. Pull up the crossed bars. ([Picture O](#))
- g. Press down the snap release of the hub to take down the rear wheel. ([Picture P](#))
- h. The retracted wheelchair frame can be now put into a smaller space. ([Picture Q](#))
- i. Reversing the procedure, return the wheelchair to its usual state.



Picture A



Picture B



Picture C



Picture D



Picture E



Picture F




Picture G



Picture H



Picture I

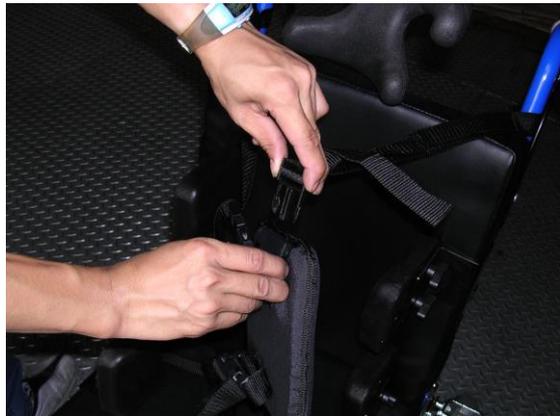

Picture J



Picture K



Picture L



Picture M



Picture N



Picture O



Picture P



Picture Q

### 3. Table Assembly

Step 1 Assemble the bracket on the table board.



Step 2 Assemble the low bracket on the chair frame.



Step 3 Fix the broad on the low bracket and fix the nuts .



Step 4 Adjust the width and comfortable position then fasten the bolts.

