

**Impotence: (Erectile dysfunction):** it is inability to achieve or maintain a hard and a rigid Penis sufficient for sexual Intercourse

**Causes:**

- \* **Psychogenic**
- \* **Diabetes mellitus**
- \* **Venous leakage**
- \* **Post operative complications**
- \* **Kidney disorders**
- \* **Spinal cord injuries**
- \* **Complications of certain drugs**
- \* **Old age**
- \* **Some different causes**
- \* **Unknown**

**ACTIVE TREATS ALL CAUSES ABOVE**

**ACTIVE is European technique with permanent guarantee**

**Mechanism of Action**

**Active depends on creating negative pressure that makes the blood passing through the penis and cause an erection .**

**Advantages :**

- \* **Safe and effective 100%.**
- \* **Easy to use.**
- \* **With out side effects.**
- \* **Maintains erection for 30 minutes.**
- \* **Effective for complete and partial impotence.**
- \* **Helps premature ejaculation.**
- \* **Enlarging the size of penis.**

**Attention : Be aware! There are ungenune devices, ineffective and no health approvals**



## Components :

### 1 - Vacuum Pump



### 2 - Transparent cylinder



### 3 - Two Loading rings and conical loader



### 4 - Multiple Size rubbers ( Small - Medium & Large )



### 5 - Water soluble lubricant.



## Connection:

1 - Insert three alkaline batteries ( 1.5 volts ) size AAA inside the vacuum pump and close the cover. ( see figure )



2 - Connect the pump in to the large side of the cylinder ( see figure )



3 - The small connector with loading conical rings should be fitted in to other side .



4 - Roll the rubber at the top of the cylinder by using conical loading ring ( see figure A ), after that release the conical loading ring to insert the penis inside the cylinder ( see figure B ), if you do have large penis bigger than average, in this case use the large connector instead ( see figure C ).

Figure (A)



Figure (B)



Figure (C)




## Procedure :


1 - Put a little amount of lubricant around the base of the penis, the shaft, and glans ( see figure ).




**2 - Put your penis inside the cylinder and press tightly against the body**

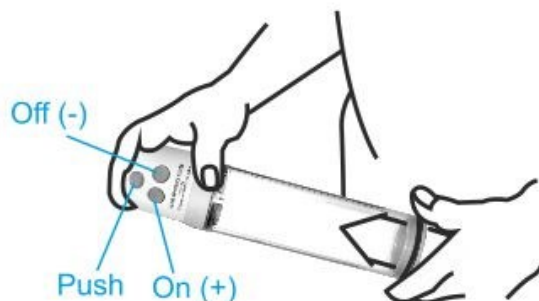

**3 - Switch On the pump, the vacuum will cause the penis growing up inside the cylinder.**

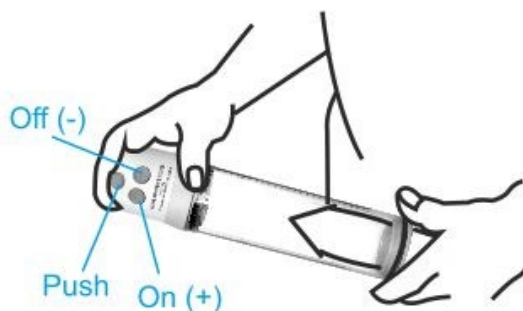


**4 - If you feel pain, press ( push ). The pain will disappear immediately ( It is not necessary to feel pain )**

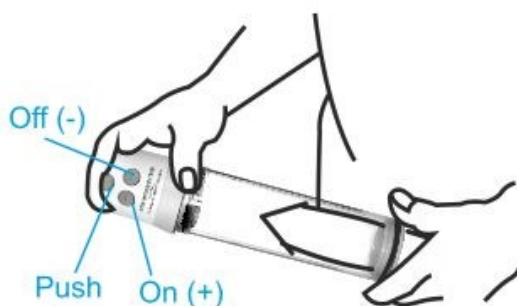


**5 - When you get hard erection inside the cylinder follow these steps :**

**A - Roll the rubber down into the base of the penis to keep the erection and make sure to push the cylinder tightly towards abdomen to prevent leakage of vacuum**

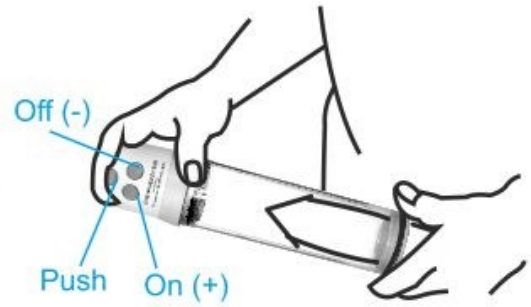


**B - After releasing the rubber switch Off the pump.**

**C - Remove the device by pressing ( Push )**

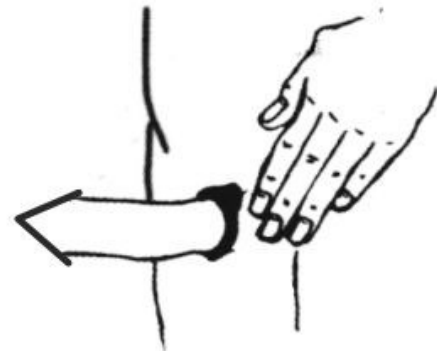

**D - When you finish sexual intercourse, remove the rubber to make the penis flaccid as before ( see figure )**

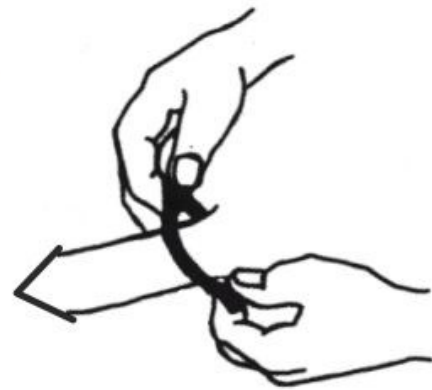




## The rubbers:

- 1 - Select the suitable rubber ( the small for standard and the large for large penis ).
- 2 - Do not sleep with rubber maintained on your penis
- 3 - Keep the rubber at the base of penis during sexual intercourse.
- 4 - To get hard erection use two rubbers at the same time.
- 5 - In some cases, the small rubber may prevent passing of seminal fluid in a good manner, the seminal fluid will be passed when removing the rubber.
- 6 - Lubricant should be used to ease releasing the rubbers.
- 7 - Keep the rubber ( do not through it away ) and wash after use.

### Notes :

- \* Doctors and experts are recommending users to have more training to use Active for ( 5 ) days period without rubbers just to activate blood vessels and penis corporal bodies responsible for erection. The users can use the rubbers after sufficient training.
- \* Usually, the operation of device does not exceed one minute, in some cases it takes few minutes, later on, it takes just seconds.
- \* Active device suits all ages
- \* Well persons are allowed to increase their sexual efficiency.
- \* There is no complications for this device in short or long run.
- \* In some cases, the users can restore their potency and return normal
- \* You can use Active in any positions, standing and lying, but standing position gives the best results.
- \* All spare part are available.

- 
- \* Use good version of batteries ( Alkaline ) such as Energizer, Kodak or any similar long life to give the best results.**
  - \* There is no complication of lubricant on you or your partner, use the lubricant KY gel, Aqua gel or any similar gel prescribed by physician or pharmacist.**
  - \* To get an excellent result shave the base of penis**

### **Washing device**

- 1- Wash all parts with tap water and soap.**
- 2 - Do not expose the pump to the water but wipe it by wet cloth.**
- 3 - Dry all parts in a good way and return them back to the bag.**
- 4 - Do not use any antiseptic solutions as alcohol, etc.. , because it may hurt and damage the device and rubbers**
- 5 - Try to remove the batteries when you leave the device more than one week.**



# Guarantee Card

No ( )

Date:

Name :

- \* Easy to use - Acts with normal battery - Without surgery - Without drugs
- ACTIVE** has low price with more safety and effectiveness

## Important notes :

- \* Bring this guarantee with every visit
- \* The validity of maintenance is for ever except fracture and water exposure
- \* Maintenance dose not include the spare parts such as batteries, Labricant and rings
- \* Keep the vacuum pump dry and do not expose to water

Visits  1  2  3  4  5  6  7  8  9  10



**The: Exclusive agent in Middle East and Arab Countries For Active Impotence System**

Tel: (00962 6 ) 5656439 - 5656479 - Fax (00962 6 ) 5656439

Mobile: 0797280814 - 0797280813 - 0796816482

Ministry of Interior circle - Hassoneh Bldg. 2nd Floor Office No.211, Jabal Al hussein, Amman-Jordan

[www.mawal-active.com](http://www.mawal-active.com)

E-mail: [info@mawal-active.com](mailto:info@mawal-active.com)