

POSTURAL AIDS IN SITTING POSITION

SYST'AM® P951D / BACK SUPPORT & THERAPEUTIC COMBI-PACKS

BACK SUPPORT MADE OF VISCOELASTIC FOAM WITH MEMORY EFFECT AND MULTI-BEARING SURFACE

MATERIALS

80 kg/m³



Visco foam



Removable and breathable cover

Foam maintenance :



(Do not immerse in water)

Cleaning of the cover :




(UP TO 90°C)

INDICATIONS

P951D Back support and Therapeutic Combi-Packs can be used to :

- 1- Improve positioning in wheelchair.
- 2- Reduce forward sliding.


SYST'AM® P951D

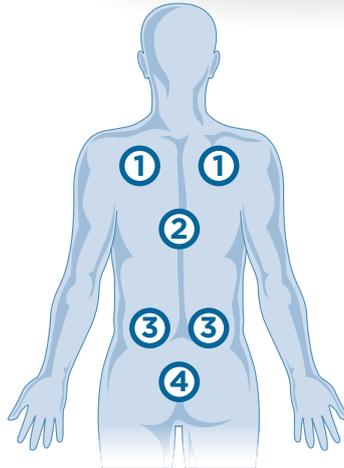


- ① SHOULDER BLADES / SHOULDERS
- ② VERTEBRAL ZONE
- ③ LUMBAR ZONE
- ④ SACRUM ZONE




WITHOUT BACK SUPPORT CUSHION

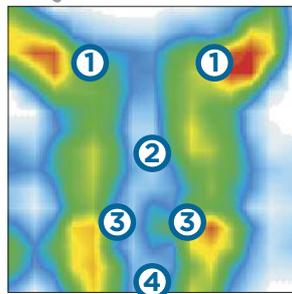

WITH REGULAR FOAM BACK SUPPORT

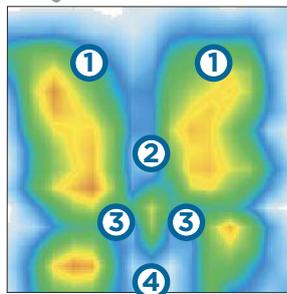

WITH SYST'AM® BACK SUPPORT CUSHION

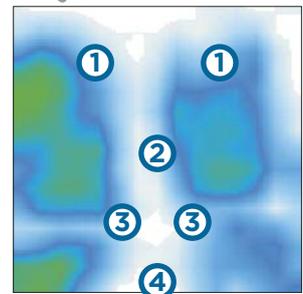

Item code	Designation	Sizes (W x L x H) (cm / in)
P951DS1HW	P951D - Back support Small	44 x 39 x 9 / 17 x 15 x 3,5
P951D1HW	P951D - Back support Standard	49 x 44 x 9 / 19 x 17 x 3,5
P951DL1HW	P951D - Back support Large	49 x 50 x 10 / 19 x 20 x 4

WARRANTY

2 YEARS



SYST'AM® P951D / BACK SUPPORT & THERAPEUTIC COMBI-PACKS

PRODUCT «ADVANTAGES»

SYST'AM® BACK SUPPORT IS WITH THE CUSHION FOR THE PREVENTION OF PRESSURE SORE :

→ A VERY COMPLEMENTARY SYSTEM

- ↘ Provides significant relief from the spinal pressure points.
- ↘ Reduces the risk of sores forming in the sacrum area due to the reduction in sagittal instability (forward slipping).
- ↘ Reduces shear effects on the sacrum/buttock area thanks to the better pelvic support.



AN ANATOMIC FORM

- ↘ Respects the morphology of the back resulting from syphosis.
- ↘ Respects the Anatomy of the low lumbar, an important aspect of posture.

IMPROVED COMFORT OF THE PATIENT IN WHEELCHAIR

- ↘ Side padding to protect the axillary-dorsal zone from contact with the frame of the chair, often painful for wheelchair users.

PERFORMANCE OF THE MULTIBEARING CONCEPT



- ↘ The multibearing surface enables a better pressures distribution through the transfer of pressures from areas with low bearing capacity toward areas with high bearing capacity :

WHITE AREAS :

Areas with low bearing capacity (gentle support) frees vertebral zone from pressure peaks and favors proper dorsal fit.

BLUE AREAS :

Areas with high bearing capacity (firmer support) transfers pressure from high-risk areas and accentuates lateral stability.

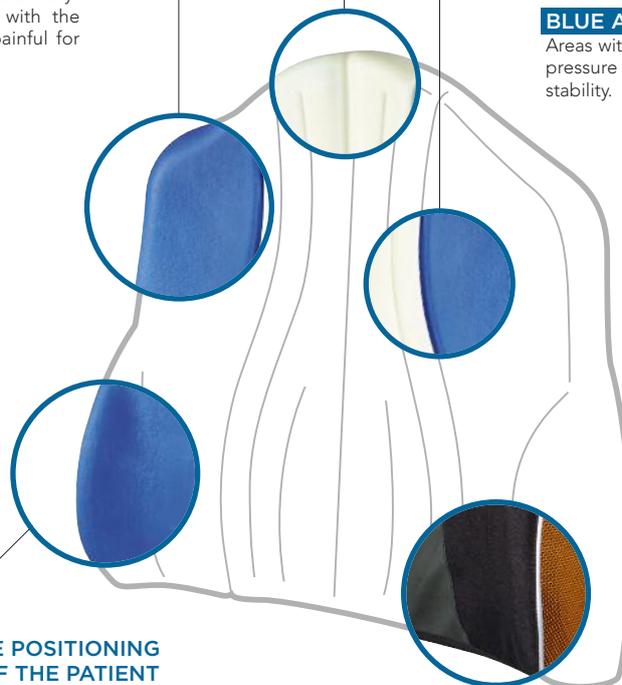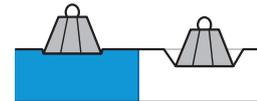

IMPROVEMENT OF THE POSITIONING AND THE STABILITY OF THE PATIENT

- ↘ By stabilising the pelvic area it contributes to preventing the sliding and risks of falling. This is a permanent issue for elderly at home and in nursing homes ;
- ↘ The sides "pads" reduce the lateral instability, which is frequent by handicaped.


COVER



- ↘ In the dorsal area, two-way stretch fabric coated with polyurethane that breathes and is totally waterproof.



- ↘ Stretch, coated fabric resists abrasion and constraints of intensive daily use.

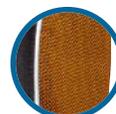

- ↘ Highly resistant technical fabric positioned in areas subject to greater abrasion.



UNIVERSAL ATTACHMENT SYSTEM

- ↘ Enables adaptation of the back support to most wheelchair models




POSTURAL AIDS IN SITTING POSITION

SYST'AM® P951D / BACK SUPPORT & THERAPEUTIC COMBI-PACKS


PRODUCT «ADVANTAGES»

SYST'AM® THERAPEUTIC PACKS


CUSTOMISED YOUR THERAPEUTIC PACK WITH THE CUSHION OF YOUR CHOICE.

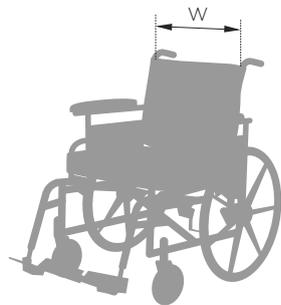


WIDTH OF WHEELCHAIR

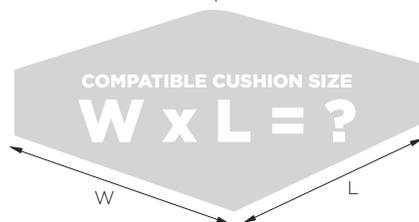

BACK SUPPORT Small P951DS BACK SUPPORT Standard P951D BACK SUPPORT Large P951DL

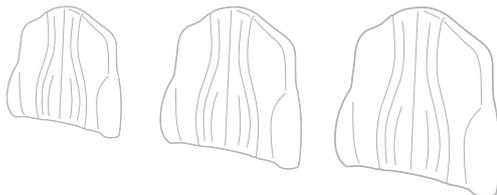

CONSULTS THE COMPATIBILITY CHART «CUSHIONS/BACK SUPPORTS»

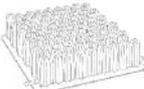-  HR-FOAM-KISSEN P371C
-  SYST'AM GEL / SYST'AM DUOGEL® P311C
-  POLYFORM® Mixte P314C
-  DUOFORM® P341C
-  VISCOFLEX® P361C
-  VISCOFLEX®+ mit Abduktionskeil P361CP
-  VISCOFLEX®+ P361CA
-  POLYAIR® 60 P301C
-  POLYAIR® 100 P302C



SYST'AM® P951D / BACK SUPPORT & THERAPEUTIC COMBI-PACKS

PRODUCT «ADVANTAGES»

WHEELCHAIR WIDTH (cm)	30/32	34/36	38		40		43/44		45		48/50		55	60						
AVAILABLE SIZES (cm)	32 x 32 cm	36 x 36 cm	38 x 38 cm	38 x 43 cm	40 x 40 cm	40 x 43 cm	40 x 46 cm	42 x 42 cm	42 x 46 cm	43 x 43 cm	45 x 42 cm	45 x 43 cm	45 x 46 cm	46 x 40 cm	46 x 46 cm	48 x 43 cm	51 x 46 cm	51 x 50 cm	56 x 46 cm	61 x 50 cm

OVERVIEW OF CUSHIONS SIZES

	●																			
	●																			
	●																			
	●																			
	●																			
	●																			
	●																			
	●																			
	●																			
	●																			

SIZES COMPATIBILITY «CUSHIONS/BACK SUPPORTS»

	+																			
	+																			
	+																			